

# Star White

---

**A Patient Interactive Dental  
Whitening System**

**Designed for Clinical  
Success!**

***Temrex Corporation The Right Choice!***

# Star White Dental Whitening Ensemble



**Temrex Corporation The Right Choice!**

# You Can Achieve Results Like This in About One Week

---



*Temrex Corporation The Right Choice!*

# Star White Ensemble

Star White Kit contains all you need to for your patient's whitening program.



8 Syringes of Star White & mixing tips.

Tray material

Interactive patient communication system featuring a simple shade system & patient instructions

*Temrex Corporation The Right Choice!*

# Patient Motivation

---

- The key to any successful whitening procedure is to motivate the patient to start and maintain the program.
- Issues such as diet restrictions & time to allocate for whitening should be discussed & solutions offered.
- Ideal time for whitening is after dinner & before bed.
- Stress need to continue on a nightly basis to achieve results.
- Ensure that the trays fit comfortably and do not impinge on the soft tissue.
- Have members of the dental team contact the patient during whitening procedure to check on compliance.

***Temrex Corporation The Right Choice!***

# First Appointment

---



Oral exam & discussion of the effects of whitening.

Note presence of any existing restorations & discuss effect of whitening on them.

Take impressions for trays

**Stress importance of patient involvement**

*Temrex Corporation The Right Choice!*

# Fabricate Trays

---



**Block out teeth to be whitened with wax.**

**Scallop gingival area of each tooth on the model to ensure good seal**

**Vacuum form & trim trays**

***Temrex Corporation The Right Choice!***



# Second Appointment

---



**Try in & fit trays.**

**Show patient how  
to place material in  
tray.**

**Review home care  
& diet**

***Temrex Corporation The Right Choice!***



# The Star System

---



**Select the tooth to monitor and mark with a blue star on the shade guides.**

**Have the patient mark improvements on shade guide with gold stars.**

**Success means easy simple monitoring & communication**

***Temrex Corporation The Right Choice!***

# Patient Communication

---

- Using the shade guide, members of the dental team can call the patient and monitor progress.
- Problems can be dealt with immediately.
- If progress is not fast enough, the dentist can increase frequency or duration.
- Communication is the key to patient compliance & a successful outcome

*Temrex Corporation The Right Choice!*

# Whitening in One Week

---



*Temrex Corporation The Right Choice!*

# Star White

## Dental Whitening Ensemble

---

- S Selection:** Selecting the proper shades & educating the patient.
- T Trays & Training:** Fabricating trays and training your patients to use them.
- A Action:** Getting your patient started on a daily program.
- R Review & Reassess:** Checking the results within 2 weeks.

*Temrex Corporation The Right Choice!*

# Product Information

---

## Star White Dental Whitening Ensemble

**Complete Kit with 8 syringes, tray material and patient education information.**

**Temrex Corporation**

**Tel: 1-800-645-1226**

**E Mail: [temrex@centurytel.net](mailto:temrex@centurytel.net)**

**Web Site: [www.temrex.com](http://www.temrex.com)**



***Temrex Corporation The Right Choice!***